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CareEvent

The screenshot shows the Philips CareEvent software interface. The title bar reads "Philips CareEvent - [Care Assignments - Memorial Hospital]". The menu bar includes "File" and "Help". The toolbar contains icons for "Care Assignments", "Event Notifications", "Users and Roles", "Mobile Devices", "Scheduled Messages", "Send Message", and "Event Reports". Below the toolbar, there is a dropdown menu for "Telemetry South" and a "Charge Nurse" dropdown menu set to "Peterson, Nancy". There are also buttons for "Unassign", "Clear All Assignments", and "Print". On the right side, there are "Confirm" and "Discard" buttons.

Name		Role(s)
	Azzarro, Luke	Care Tech
	Johnson, Elaine (15619275298)	Nurse Manager
	Perkins, Gary	Nurse, Buddy Nurse
	Peterson, Nancy	Charge Nurse, Nurse
	Smith, Jill	Nurse, Buddy Nurse
	DeGragario, Monica	Nurse, Buddy Nurse

Bed	Patient Name	Nurse	Buddy Nurse	Care Tech
TeleS1		Perkins, Gary	Smith, Jill	Azzarro, Luke
TeleS2		Perkins, Gary	Smith, Jill	Azzarro, Luke
TeleS3		Perkins, Gary	Smith, Jill	Azzarro, Luke
TeleS4		Perkins, Gary	Smith, Jill	Azzarro, Luke
TeleS5		Smith, Jill	Perkins, Gary	Azzarro, Luke

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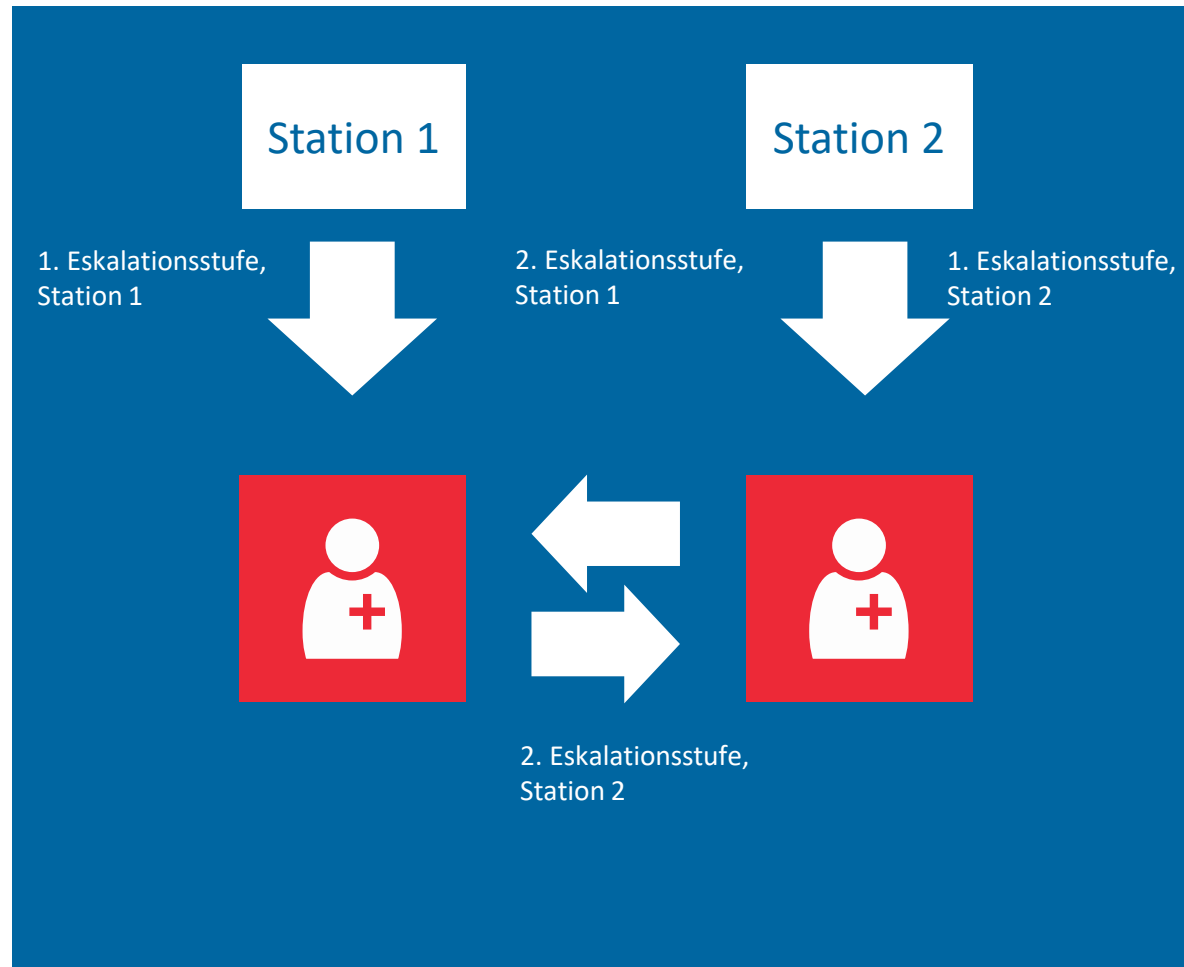
Alarmberichte

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PHILIPS Event Reports

Analysis by Alarm and Event | Analysis by Alarm and Event Notifications | Analysis By Caregiver | Analysis By Bed | Alarm Settings | Quickview

Event Reports

Analysis by Alarm and Event
 These reports provide an inventory of all alarms generated from the patient monitoring and nurse call systems.
[Overall Inventory By Unit](#)
[Top Events By Unit](#)
[Limit By Unit](#)

Analysis by Alarm and Event Notifications
 These reports provide an inventory of the alarms and events, including escalations, forwarded to caregivers for notification.
[Event to Notification + Escalation](#)
[Event Transcript Selection](#)

Analysis By Caregiver
 These reports provide data as to a caregiver's responsiveness to alarm and event notification including itemization of the data by escalation level and accepted percentage, escalated percentage, and non-responsive percentage.
[Caregiver Productivity Analysis](#)
[Caregiver Escalation Analysis](#)
[Caregiver Response Analysis](#)

Analysis By Bed
 These reports provide data to identify event activity at the bed level including total events sent for notification, the caregiver's specific action(s) for each notification, and a report that aggregates all missed events by the caregiver.
[Delivered Event Totals By Bed](#)
[Event Responsiveness by Bed](#)
[Delivered Event Detail By Bed](#)
[Missed Event Detail By Bed](#)

Alarm Settings
 These reports provide analysis for the frequency of alarm related setting changes and can be used to find ideal default values to improve workflow.
[Alarm Limit Changes](#)
[Alarm State Changes](#)
[Measurement Changes](#)

Quickview
 The Message Quickview report provides a 'quick view' of the overall message load for caregivers by product for a particular time period.
[Message Quickview](#)

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PHILIPS Event Transcript

P210: *** Vent Fib/Tach
 9/30/2014 12:48:34 PM

Institution: Reports Hospital
 Unit: PICU
 Product: Philips Patient Monitoring
 Event Category: Wib-Tach
 Result: Success

Action Time	Action	Device			
9/30/2014 12:48:34 PM	Announced	PUR24210 IM			
9/30/2014 12:48:34 PM	Notified	2UR329INPH			
Action Time	Action	Caregiver	Role	Level	Device
9/30/2014 12:48:35 PM	Delivered	Reed,Sandyr	Nurse	First	15413275238
9/30/2014 12:48:35 PM	Read	Reed,Sandyr	Nurse	First	15413275238
9/30/2014 12:48:37 PM	Escalated	Reed,Sandyr	Nurse	First	15413275238
9/30/2014 12:48:37 PM	Escalated	2UR329INPH			
9/30/2014 12:48:37 PM	Notified	2UR329INPH			
Action Time	Action	Caregiver	Role	Level	Device
9/30/2014 12:48:37 PM	Delivered	Smith,Tom	Buddy Nurse	Second	
9/30/2014 12:48:38 PM	Read	Smith,Tom	Buddy Nurse	Second	
9/30/2014 12:48:39 PM	Escalated	Smith,Tom	Buddy Nurse	Second	
9/30/2014 12:48:39 PM	Escalated	2UR329INPH			
9/30/2014 12:48:39 PM	Notified	2UR329INPH			
Action Time	Action	Caregiver	Role	Level	Device
9/30/2014 12:48:45 PM	Delivered	Philips,Beth	Charge Nurse	Third	
9/30/2014 12:48:45 PM	Read	Philips,Beth	Charge Nurse	Third	
9/30/2014 12:48:48 PM	Accepted	Philips,Beth	Charge Nurse	Third	
9/30/2014 12:48:48 PM	Notification Expired	2UR329INPH			

PHILIPS Event Transcript

P210: *** Vent Fib/Tach
 9/30/2014 12:48:34 PM

The waveform graph displays multiple channels of patient data, including heart rate (HR) and respiratory rate (RR), with a prominent blue line indicating a specific event or alarm.

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