



How to read curves for NIV


Monitoring pressure, flow and volume curves during NIV
To assess common problems

- Auto triggering
- Double triggering
- Ineffective triggering
- Leakage


- The proposed interpretation of the proposed curves does not substitute the clinical expertise of the physician and/or respiratory therapist
- These suggestions therefore **do not** substitute the clinical judgment and decision of the physician and/or respiratory therapist

PHILIPS


Pressure (cmH₂O)

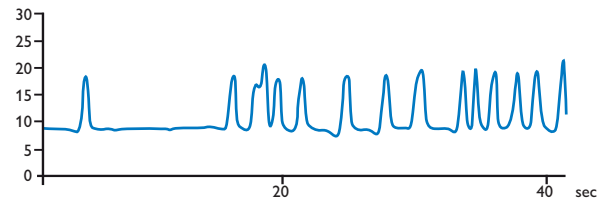

Volume (mL)

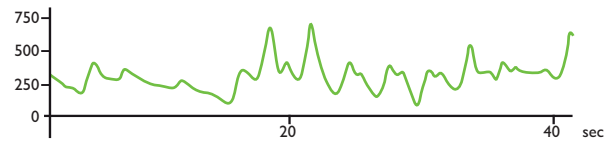


Flow (LPM)

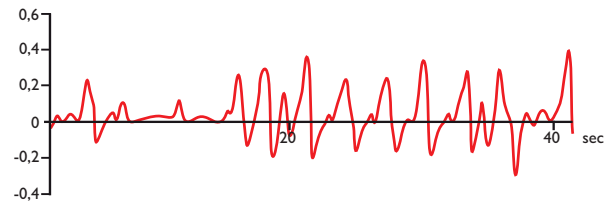

Main troubleshooting during NIV

- Auto-triggering
- Ineffective triggering
- Double-triggering
- Leakage

▼ **Complete waveform monitoring facilitates troubleshooting**

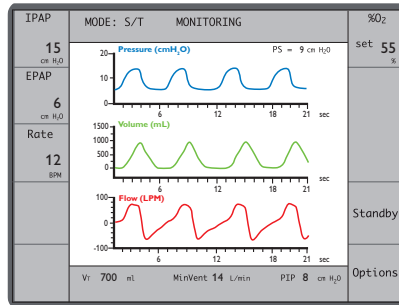


• **Pressure**

• **Volume**

• **Flow**

Two main ventilatory modes: pressure and volume mode

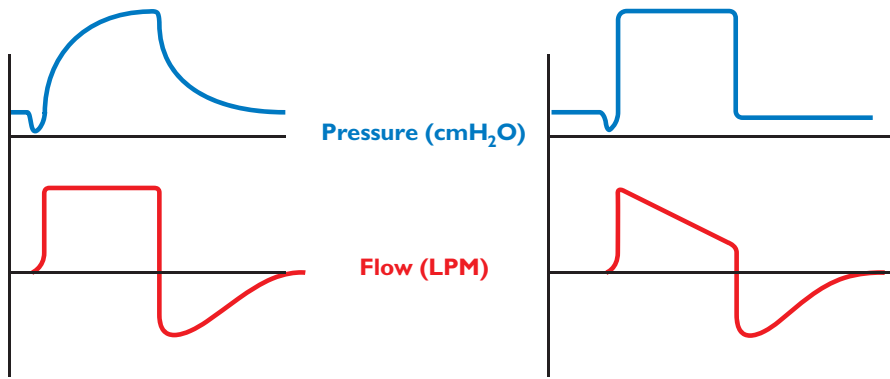


Volume mode

- Independent value is Volume
- Dependant value is Pressure

Pressure mode

- Independent value is Pressure
- Dependant value is Volume


Usual pressure modes for NIV: PCV, PSV

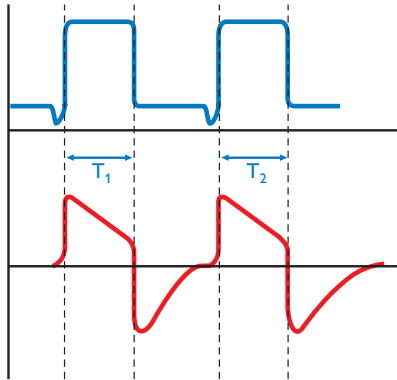

Pressure Control Ventilation (PCV)

Fixed Inspiratory Time

$$T_1 = T_2$$

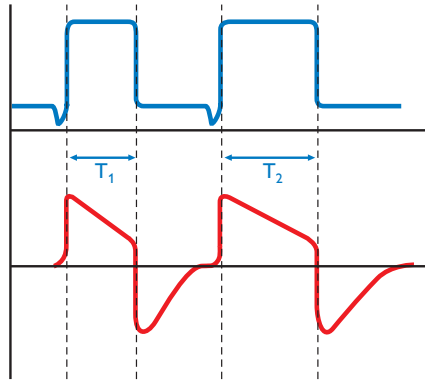

Pressure Support Ventilation (PSV)



Inspiratory Time depends on patient breathing pattern

$$T_1 \neq T_2$$

NIV requirements:


Two breaths for each of the modes

- Pressure Control Ventilation (PCV)
 - ▼ Fixed inspiratory time
- Pressure Support Ventilation (PSV)
 - ▼ Variable inspiratory times

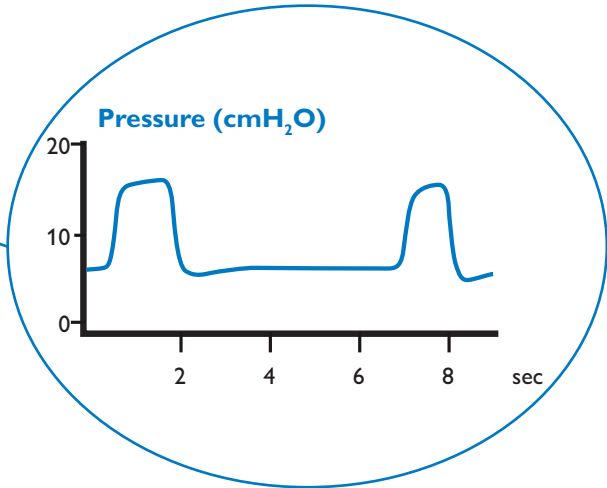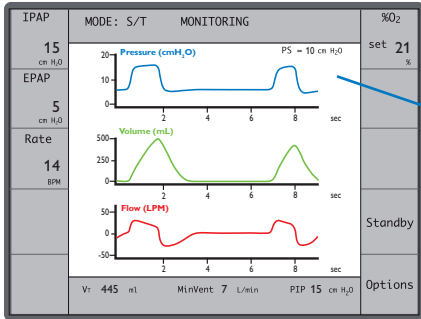

Pressure curve

Points of interest

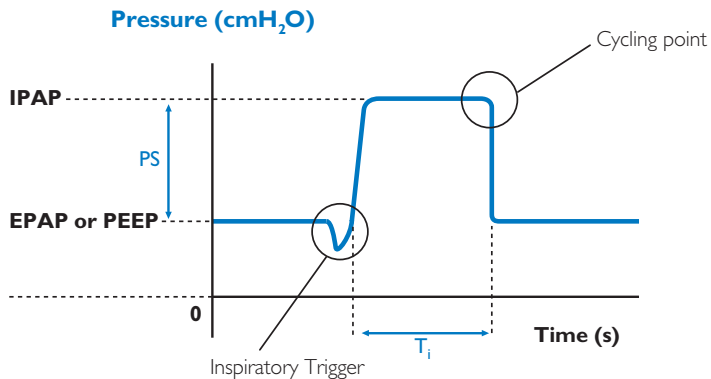

IPAP: Inspiratory Positive Airway Pressure

EPAP: Expiratory Positive Airway Pressure

PEEP: Positive End Expiratory Pressure



PS: Pressure support

T_i: Inspiratory time

EPAP = PEEP


PS = IPAP - [EPAP or PEEP]

IPAP = PS + [EPAP or PEEP]

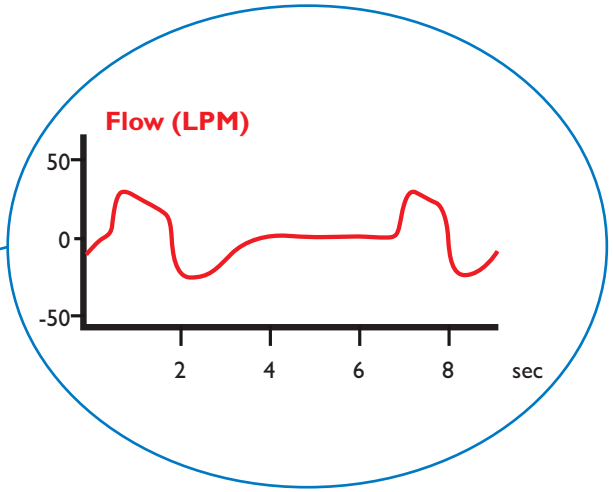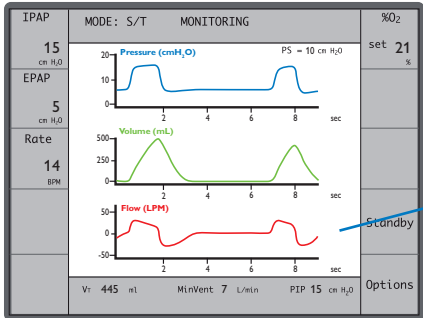

Flow curve

Points of interest

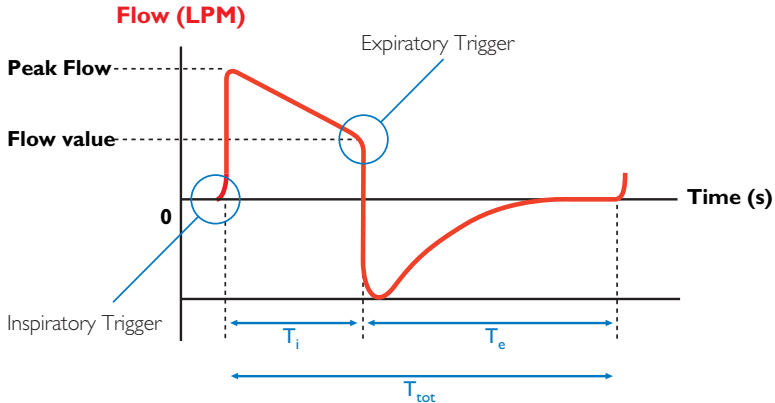

Expiratory Trigger:



- % of Inspiratory peak flow
or
- Absolute value in LPM
or
- Set inspiratory time ($=T_i$)

T_i : Inspiratory time

T_e : Expiratory time

T_{tot} : Total breath time

$$T_{tot} = T_i + T_e$$

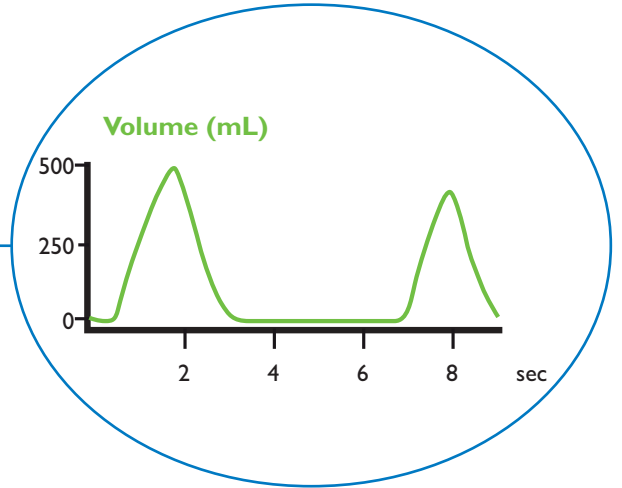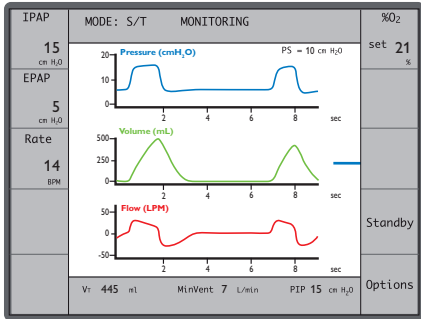

Volume curve


Points of interest

No leak
 $V_{ti} = V_{te}$


$$V_{te} = V_{ti} - V$$

Leak
 $V_{ti} > V_{te}$

Volume (mL)

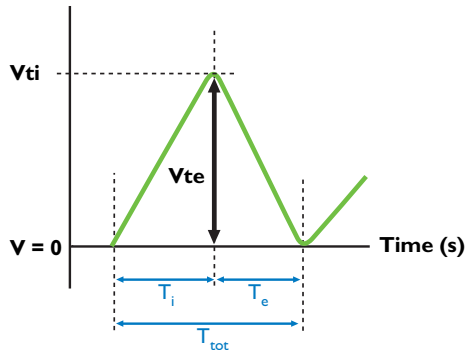

Volume (mL)

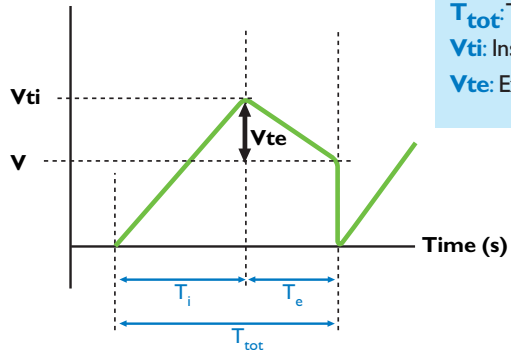



T_i : Inspiratory time

T_e : Expiratory time


T_{tot} : Total time $T_{tot} = T_i + T_e$

V_{ti} : Inspiratory total volume

V_{te} : Expiratory total volume

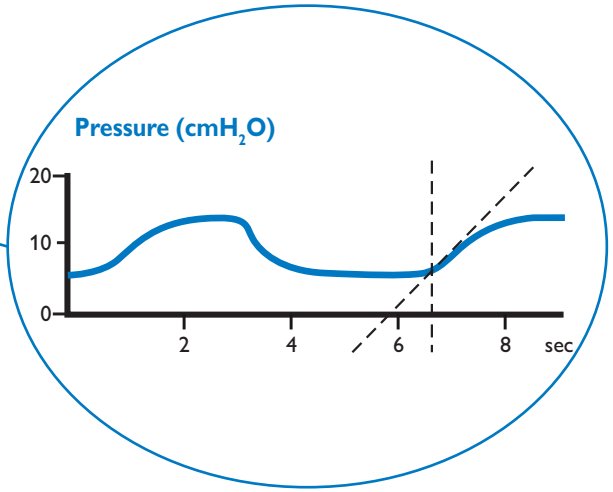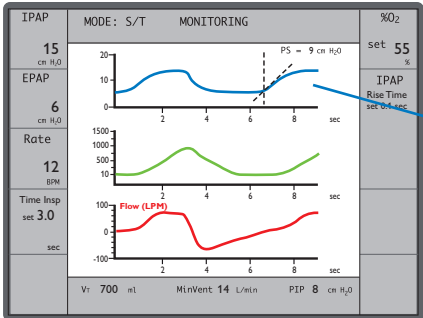


Pressure ramp ▼ Flow impact

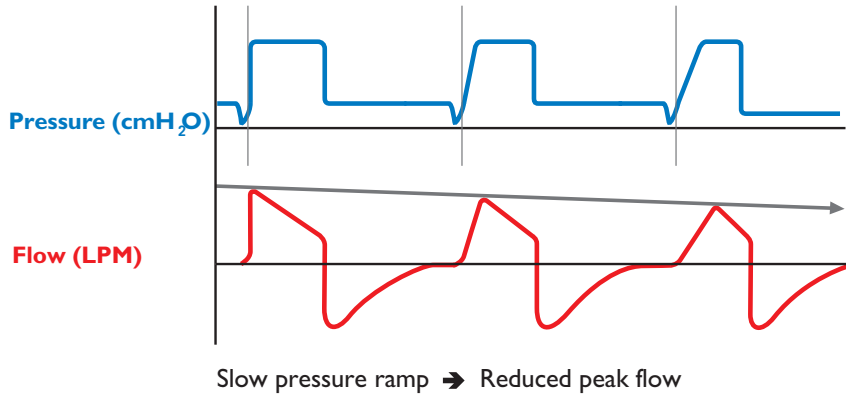


Auto triggering

Pressure (cmH₂O)

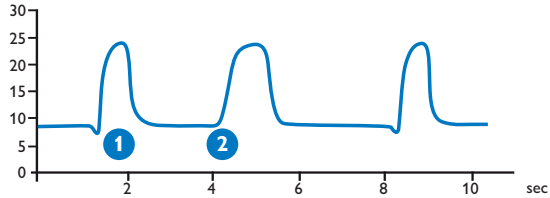


- 1 Breath normally triggered
- 2 Auto-triggered breath

Volume (mL)

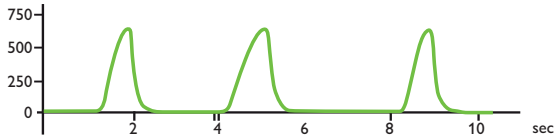

Flow (LPM)

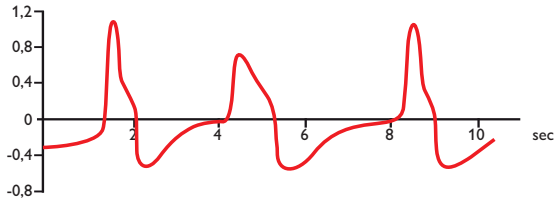


Auto triggering

Breath number 2 is triggered in the absence of patient's inspiratory effort (no abrupt decrease in pressure). This does not represent an automatic change to backup mode.

Autotriggering can occur:

- leakage
- mobile condensation in the circuit
- trigger too sensitive

Double triggering

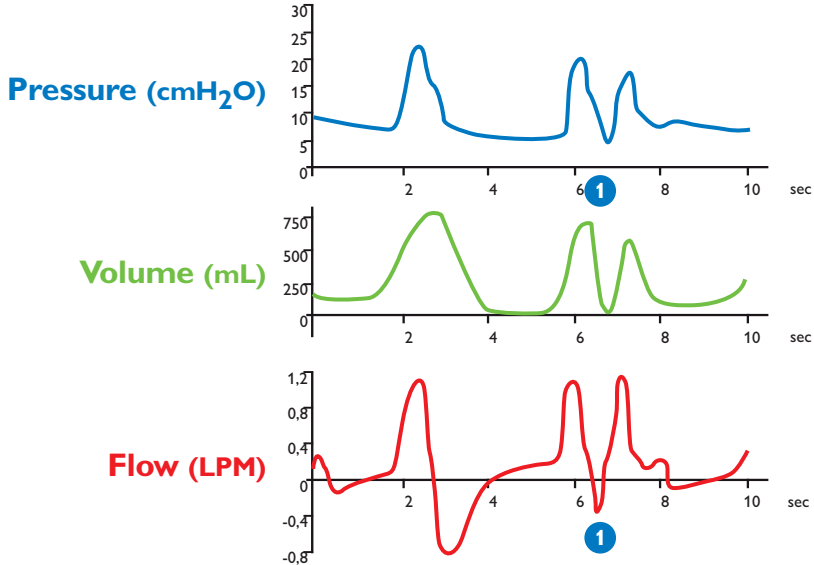

1 Double triggering

Double triggering

Double triggering defined as two consecutive ventilator cycles separated by an expiratory time less than one-half the mean inspiratory time*.


Double triggering can occur:

- ventilator inspiratory pattern ...
- patient's effort ...
- ventilator settings (insp. time, cycling criteria, ...)


*Thille et al: Patient-ventilator asynchrony during assisted mechanical ventilation. Intensive Care Medicine (2006) 32:1515-1522

Ineffective triggering


Pressure (cmH₂O)

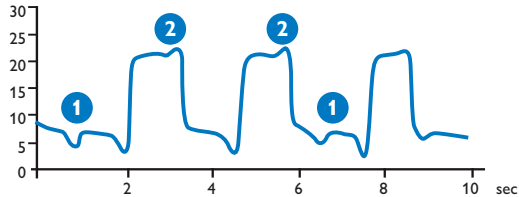

Volume (mL)

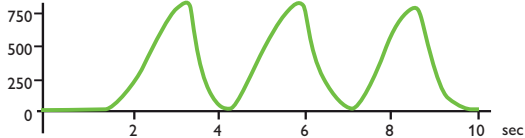

Flow (LPM)

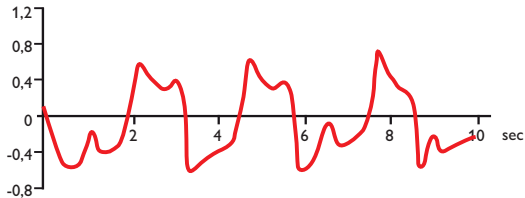

- 1 Inspiratory ineffective triggering
- 2 Expiratory ineffective triggering

Ineffective triggering

A wasted effort is defined as an airway pressure drop simultaneous to a flow increase not followed by a ventilator cycle*.



Due to patient's active expiration an increase in pressure was not recognized as end inspiration.

Ineffective triggering can occur:


- patient respiratory system mechanics
- inadequate setting (insp trigger, cycling criteria, ...)
- excessive inspiratory support

* Thille et al: Patient-ventilator asynchrony during assisted mechanical ventilation. Intensive Care Medicine (2006) 32:1515-1522

No leakage

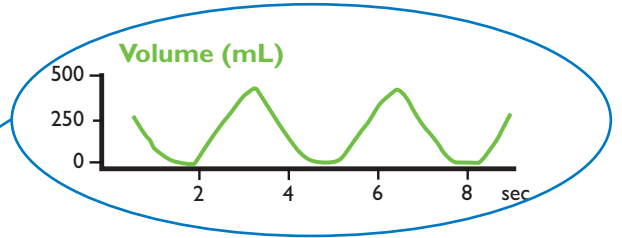
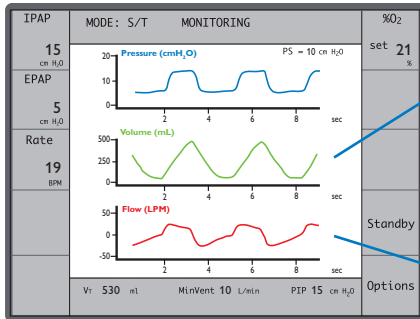


- End of expiratory volume reach 0 value

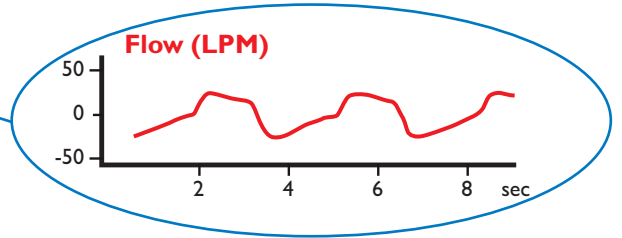


- Flow curve present no leakage readjustment

Leakage

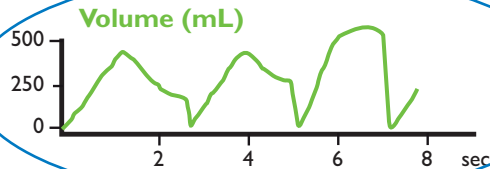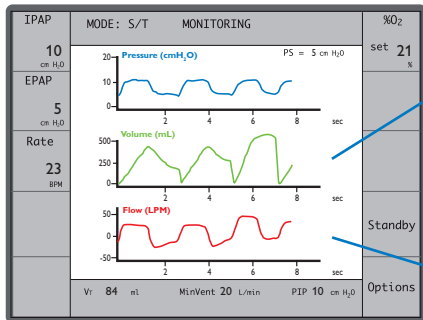


- **Volume curve drop suddenly to 0 when breath is triggered**

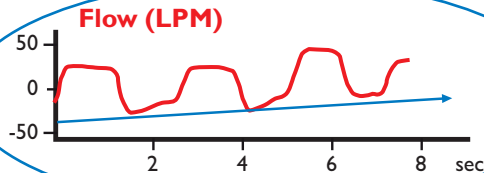

- **Base line flow is different than 0 during leakage readjustment**

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Notes:



Philips Healthcare is part of Royal Philips Electronics

How to reach us

www.philips.com/healthcare
healthcare@philips.com
fax: +31 40 27 64 887

Customer Service
+1 724 387 4000
800 345 6443 (toll free, US only)

Respironics Europe, Africa, Middle East
+33 1 47 52 30 00

Respironics Switzerland
+41 62 745 1750

Respironics Asia Pacific
+852 3194 2280

Respironics Australia
+61 2 9666 4444

Respironics France
+33 2 51 89 36 00

Respironics Italy
+39 03 62 63 43 1

North America
+1 425 487 7000
800 285 5585 (toll free, US only)

Latin America
+55 11 2125 0744

Europe, Middle East, Africa
+49 7031 463 2254

Asia
+852 2821 5888

© 2009 Koninklijke Philips Electronics N.V.
All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Printed in France.

RIT HPV 012 E * JAN 2009

